

What are Therapy Insights


Therapy Insights gives clinicians objective data to make informed decisions, providing you more information on Curve Analysis, Adherence Summary and Titration Summary.



Curve Analysis

Allows clinicians to review the most recent therapy flow, pressure, and/or flow waveforms.


- Review the last therapy's real time flow and pressure curves.
 - Uncheck either pressure or flow to increase the size of waveform.
- Use the range sliders on the bottom to zoom into a specific section of the therapy.

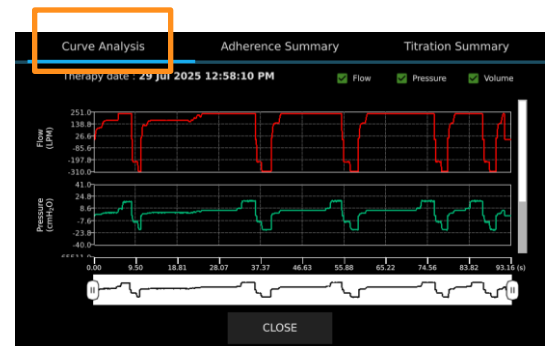

Adherence Summary

Visual summary of usage patterns


- Provides a therapy adherence summary in a matrix form for the last 7 days.
- Data represents number of therapies performed (dots) in part of a day and cumulative number of cycles for each day at the top.

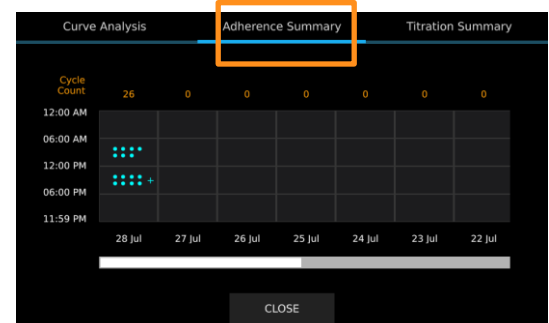

Titration Summary

A summary of the titration feedback

- Provides a horizontal bar graph for each cycle with pressure value for each phase.
- The feedback from the user (thumbs up/down/no response) correlates to the check/cross/empty mark against each cycle.
- A summary of the applicable cycles (excluding thumbs down/crossed ones) are shown in a table so a clinician decision can be made based on the min, max, and avg. pressure data.
- Titration Summary only shows feedback on the last manual mode therapy.

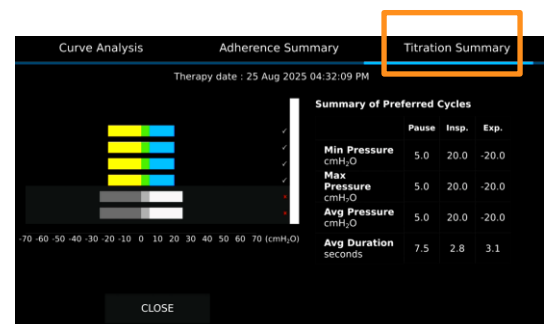


How to access Therapy Insights

There are 2 ways to access:


1. After therapy is complete

Press "View Therapy Insights"

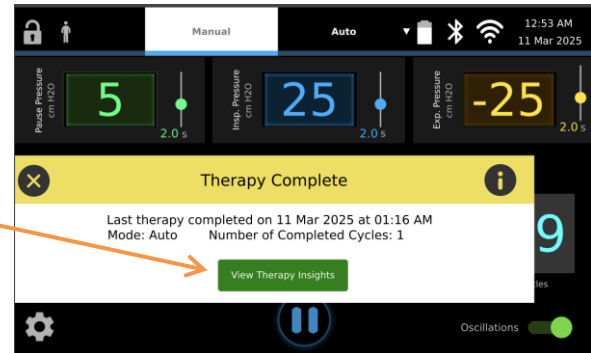

2. From Settings

1. Press "Settings" icon

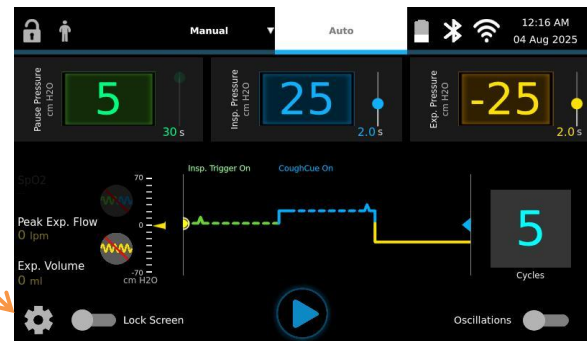

2. Press "Device Settings"



3. Press "Therapy Insights"

